

# Kangaroo ePump™

## Enteral Feeding Pump

### Loading & Rinsing of Feeding Pump

**Do not use feeding sets for longer than 24 hours.**

#### Proper loading & unloading of a feeding set



1) Flead & flush valve assembly, 2) Feed only valve assembly, 3) Distal end of feeding set, 4) Pump



1) Grasp thumb tab on valve and insert firmly into left pocket, ensure valve is fully seated. Tab should align with raised white line on left.




2) Grasp black ring retainer, gently wrap tubing around rotor, and insert retainer directly into right pocket. **Do not overstretch tubing.**



3) To properly unload a pump set, power the pump off and wait 10 seconds before removing the feeding set.

#### Proper rinsing of the feeding set

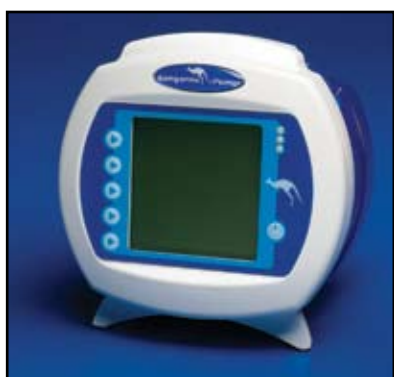

1) Disconnect feeding set from patient before rinsing. The feeding set must remain loaded during the rinsing procedure.



2) **CAUTION-** if set is removed from the pump, it may damage the set and prevent use and will negatively effect the system accuracy.



3) Empty contents of feeding bag.



4) Fill the feed bag with water to approximately half of the total capacity. Properly seal the bag and shake for 5 to 10 seconds.



5) Empty the feed bag and repeat the last step until the bag contains clean water.



6) Power the pump on. Select "Keep Settings" to continue with previous setting or "Clear Settings" to start with programming new pump settings.




7) Press the "Prime Pump" button to enter the "Prime Pump" screen.



8) Press and hold the "Hold to Prime" button until only clean water flows through the tubing. Once the rotor starts, this takes between 1 and 2 minutes.



9) Empty bag of water. Prime to remove any remaining water out of the feeding set.

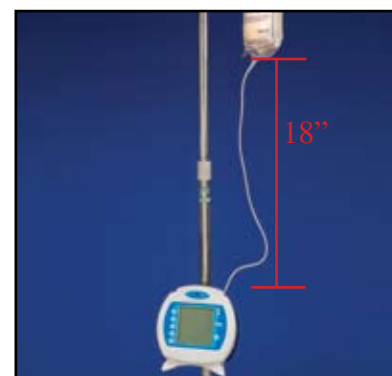

10) **CAUTION-** To achieve proper accuracy, the bottom of the feeding set bag must be 18" above the top of the feeding pump.